

Louisiana Office of Public Health Laboratories	
Test Name	Neonatal 17 α -OH-progesterone
PHL Location	Central Laboratory 1209 Leesville Avenue Baton Rouge, Louisiana 70802
CPT Code	83516
Synonyms	17-OHP
Brief Description of Test	Quantitative determination of human 17 α -OH-progesterone in blood specimens dried on filter paper as an aid in screening newborns for congenital adrenal hyperplasia (CAH).
Possible Results	Normal; and Abnormal
Reference Range	Weight <1250 grams <150ng/mL=Normal; Weight \geq1250 grams <2250 grams <90 ng/mL=Normal Weight \geq2250 grams <40ng/mL=Normal
Specimen Type	Neonatal Dried Blood Spot
Specimen Container(s):	Shipping Envelopes
Minimum volume accepted:	Minimum of 2 completely filled blood spot circles
Collection Instructions	Blood Specimens should be taken directly from a heel prick onto filter paper and, allowed to dry for at least 3 hours in a horizontal position.
Storage and Transport Instructions	Storage of samples in an environment with elevated temperatures and humidity increases the risk of false positive screening results. Transport or mail the specimen to the laboratory with 24 hours of collection.
Causes for Rejection	Specimen > 14 days old, clotted or layered, serum rings, scratched or abraded, insufficient quantity for testing, not completely dry before mailing, blood applied to both sides of the filter paper, diluted discolored or contaminated, collection using capillary tubes containing EDTA, >12 months old, circles not completely filled.
Limitations of the Procedure	Samples spot not uniformly saturated with blood - sample spots punched too close to the edge of the blood spot - poorly collected and improperly dried specimens - non-eluting blood spot due to deterioration of sample caused by exposure to heat and humidity - contamination of blood spot filter paper with fecal material Heterophilic antibodies in the sample may interfere with the assay (16,17). Hematocrit values in the blood sample may affect the measured hTSH concentration
Interfering Substances	Icteric (unconjugated bilirubin \leq 342 μ mol/L, equivalent to 20mg/d and conjugated bilirubin \leq 237 μ mol/L., equivalent to 20 mg/dL), lipemic (Intralipid <300 mg/dL) and hemolytic (additional hemoglobin \leq 0.5g/dL) specimens do not interfere with the assay. Specimens containing EDTA up to 12.5mg/mL blood, Na-citrate up to 0.0645 mol/L blood or Li-heparin up to 1.0mg/dL blood do not interfere with the assay.
References	GSP Neonatal 17 α -OH- progesterone kit package insert
Additional Information	N/A
Release Date	11/2013
Warning: If you have printed a copy of this information please be advised that the Louisiana Office of Public Health Laboratories website and methods are updated on a regular basis. Please check the on-line version of this document to ensure you are relying on the most recent release.	